



GUIDELINES FOR NEUTRALIZING VOSH EYEGLASSES

1. Discard broken frames, badly scratched lenses, lenses only, and prescription sunglasses.
2. Discard prescriptions with greater than 0.75 diopters difference between each eye.
3. Discard prescriptions with greater than 0.75 diopters cylinder.
4. Discard prescriptions with prism.
5. Write all prescriptions in minus cylinder form.
6. Ignore 0.25 diopters of cylinder and write as a sphere.
7. Round cylinder axis off to the nearest 5 degrees.
8. Make sure all glasses are cleaned and straightened.
9. Record right eye prescription first.
10. Put all glasses in zip lock bags.
11. Label bags with masking tape or computer labels on the top of the bag.
12. Save all scrap gold.

Send or deliver all completed eyeglasses to our drop-off locations:

Blaine Eye Clinic
12170 Aberdeen St NE
Blaine, MN 55449
763-757-7000

PineCone Vision Center
2180 Troop Drive
Sartell, Mn 56377
320-258-3915